



Spare Parts WG Dental oil-less air compressor


Document Version 1.02 – December 2011



Accessories and Spare Parts

(1)	Compressor system with single head		
	WG series	φ8 PU pipe	3 (3m)
		08-02 quick joint	1 set
		φ8-G1/4 quick joint	1
		G1/4□-G1/2 screw joint	1
(2)	Fuses for WG	15A	2


Electrical Circuit

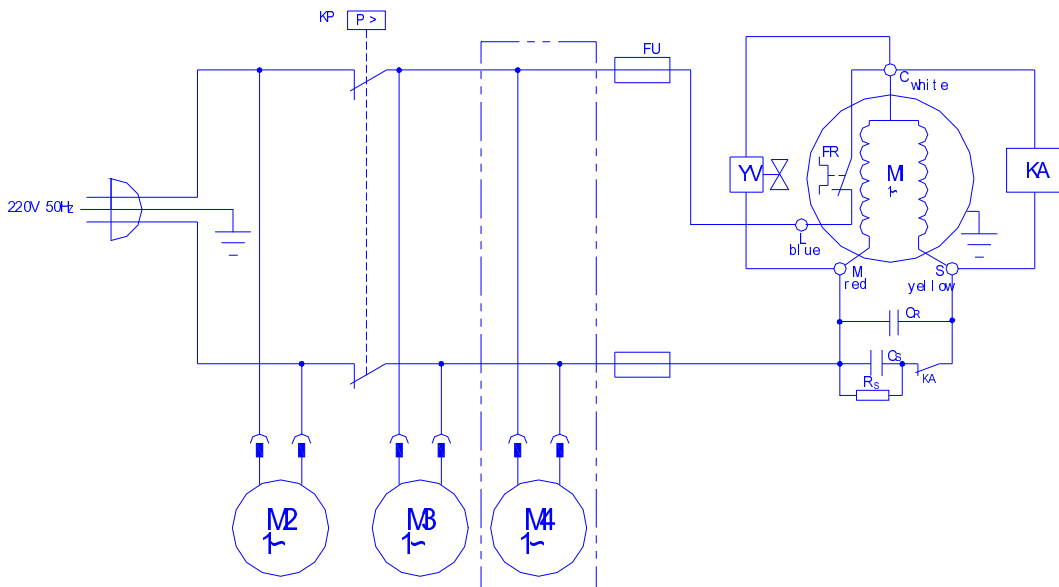

The electrical circuit


Air passage Drawing

(1) Single head machine:

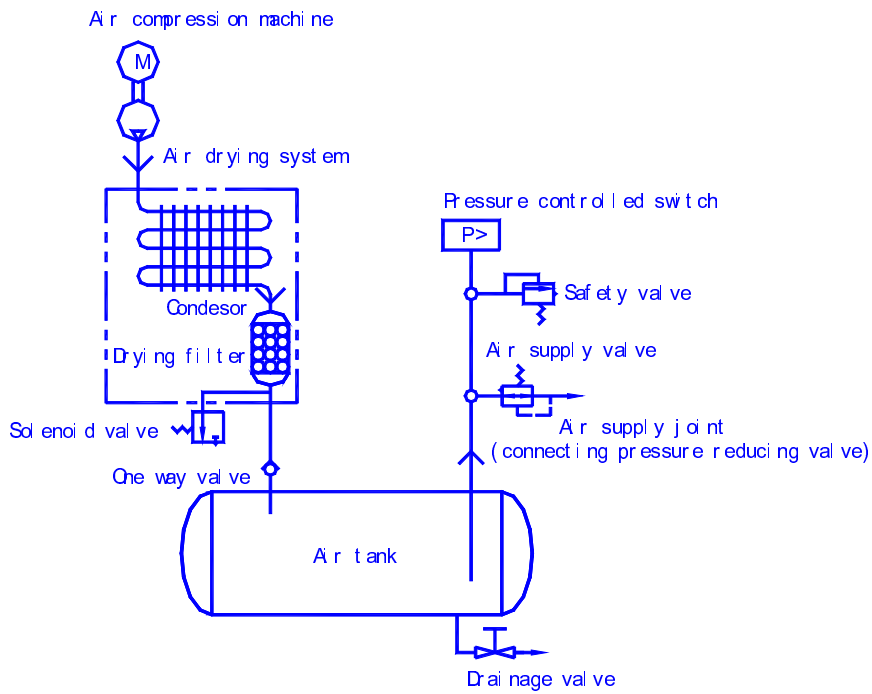

The air current drawing


The Drawing of the Compressor Structure (Version 1)

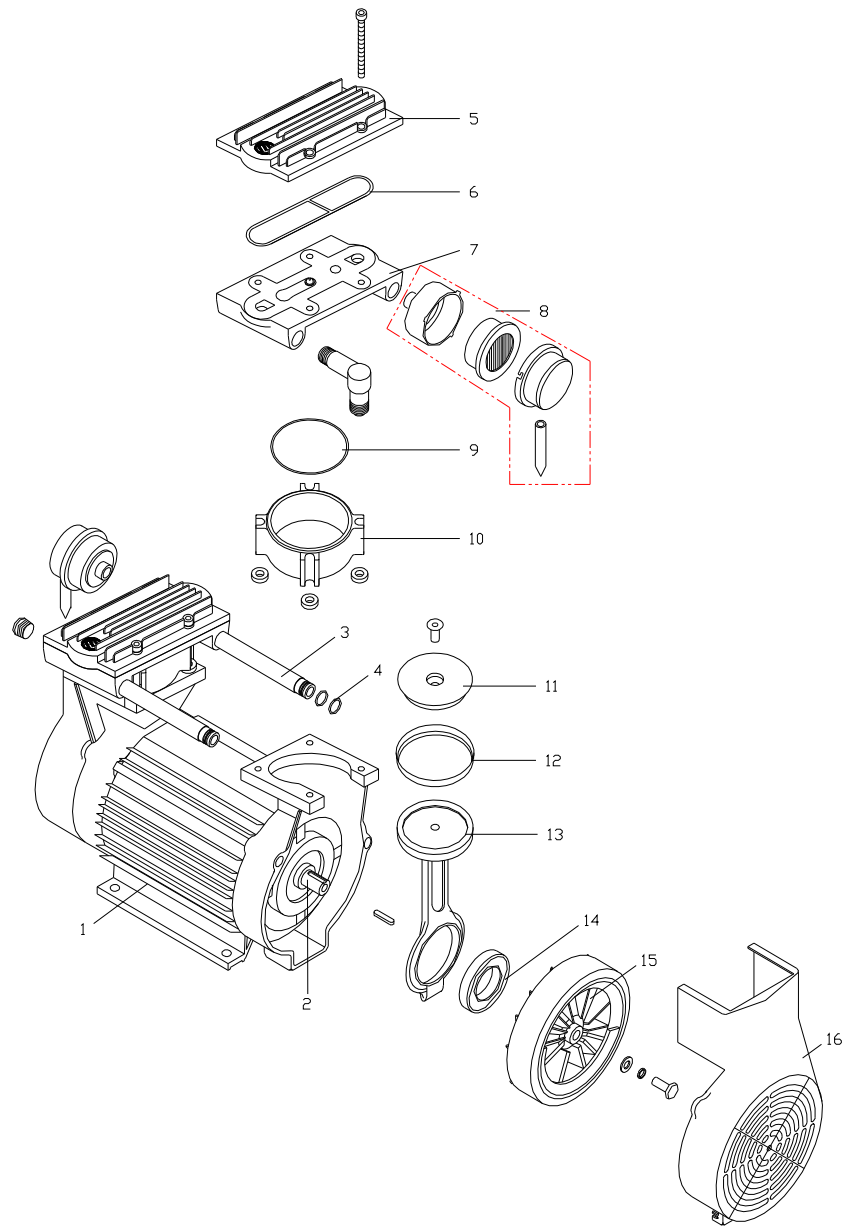

The drawing of the compressor

- 1. Motor 2. Rotor Spindle 3. Windpipe 4. Ring 5. Cylinder cover 6. Sealing 7. Cylinder Head
- 8. Air filter (refill) 9. Rubber Ring 10. Cylinder body 11. Fitting 12. Piston ring 13. Link
- 14. Bearing 15. Eccentric 16. Casing


The Drawing of the Compressor Structure (Version 2)

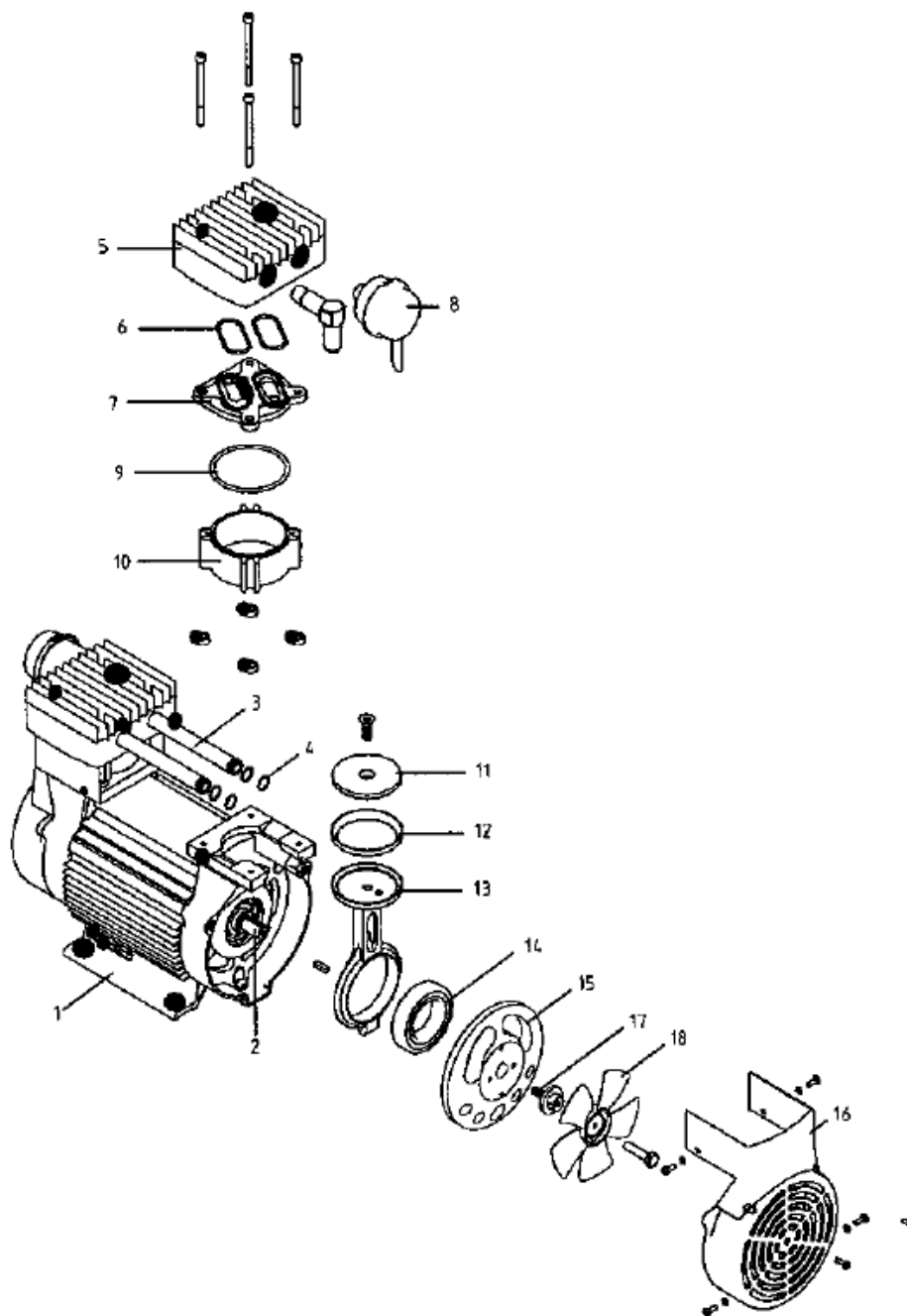

Position	Part number	Description
1	KW185-30.950	MOTOR
2	KW185-30.005	ROTOR SPINDLE
3	KW185-00.006	WINDPIPE
4	KW185-	RING
5	KW185-A40.001B	CYLINDER COVER

6	KW185-A00.004	SEALING
7	KW185-A20.001	CYLINDER HEAD
8	KW185-00.008	AIR FILTER
9	KW185-00.003	RUBBER RING
10	KW185-00.005	CYLINDER BODY
11	KW185-10.001	FITTING
12	KW185-10.002	PISTON RING
13	KW185-10.003	LINK
14	KW185-NSK6009V	BEARING
15	KW185-A00.001	ECCENTRIC
16	KW185-00.011	CASING
17	KW185-00.013	FAN CONNECT
18	KW185-00.012B	FAN

The drawing of the system

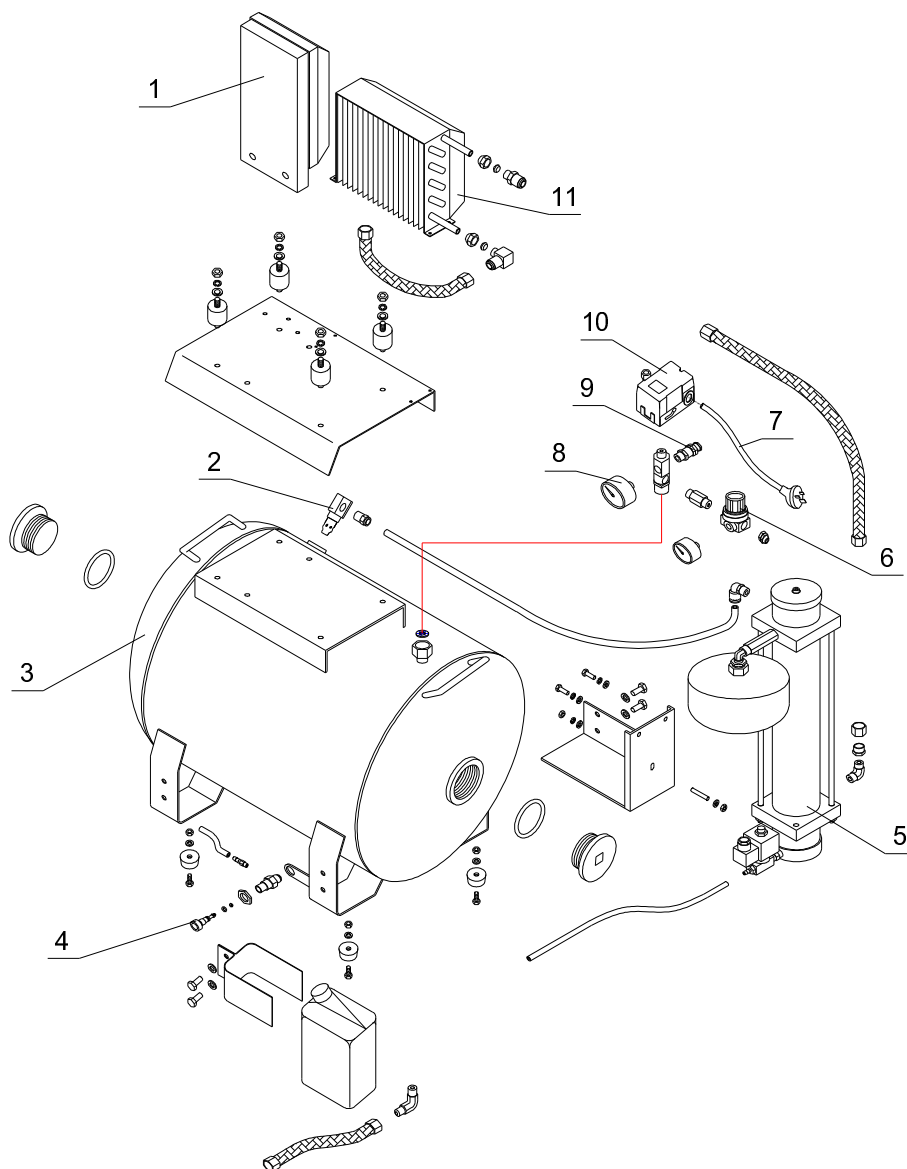

- 1. Electrical switch box
- 2. Retaining valve
- 3. Air tank
- 4. Drainage valve
- 5. Air-drying system (order part for customer)
- 6. Air supply valve
- 7. Power cable
- 8. Barometer
- 9. Safety valve
- 10. Pneumatic control switch
- 11. Condenser (order part for customer)